



<p><b>GB</b>                      <b>INSTRUCTION FOR USE</b> <b>GREENFIT POUCHES AND REELS</b></p> <p><b>1. INSTRUCTIONS FOR USE</b> Use GREENFIT 2 - 27 pouches and reels to pack medical devices to be sterilized by steam or EO gas. GREENFIT 3 pouches and reels also can be sterilized by formaldehyde.</p> <p>Seal the open end(s) with a rotary or impulse sealing machine at the recommended temperature (150-180 °C) and following the instructions of the sealing machine manufacturer.</p>	<p><b>2. ESTERNAL INDICATORS PRINTED ON THE PRODUCTS:</b></p> <p>GREENFIT 2 - 27: EO gas: Yellow/Brown sterilized Steam: Brown sterilized</p> <p>GREENFIT 3: EO gas: Yellow/Brown sterilized Steam: Brown sterilized Formaldehyde: Green sterilized</p> <p>The packaging is for single use only. Do not use if pack is damaged.</p> <p>Report any serious incident involving the device to the manufacturer and the competent authority.</p>	<p><b>3. STORAGE CONDITIONS</b> Keep away from direct sunlight and heat. We recommend: - Store at temperature 10-30 °C before use. - Relative humidity: 30-60 % before use. Outer cartons should not suffer physical damage in handling and storage.</p> <p><b>EN 868-5    ISO 11607-1</b></p> <p></p> <p><b>STERIFIT S.r.l.</b> Via dei Martinitt, 3 20146 Milano - Italia info@sterifit.com – www.sterifit.com</p>
---	--	---

<p><b>GB</b>                      <b>INSTRUCTION FOR USE</b> <b>GREENFIT SELF-SEALING POUCHES</b></p> <p><b>1. INSTRUCTIONS FOR USE</b> Use GREENFIT 2 - 27 self-sealing pouches to pack medical devices to be sterilized by steam or EO gas. GREENFIT 3 self-sealing pouches can be also sterilized by formaldehyde.</p> <p><b>Sealing instructions</b></p> <ul style="list-style-type: none"> <li>- Unfold flap;</li> <li>- Insert item into pouch;</li> <li>- Remove backing from sealing strip;</li> <li>- Refold along crease pressing down firmly from center to each side to form good seal between adhesive strip and paper and plastic.</li> </ul>	<p><b>2. ESTERNAL INDICATORS PRINTED ON THE PRODUCTS:</b></p> <p>GREENFIT 2 - 27: EO gas: Yellow/Brown sterilized Steam: Brown sterilized</p> <p>GREENFIT 3: EO gas: Yellow/Brown sterilized Steam: Brown sterilized Formaldehyde: Green sterilized</p> <p>The packaging is for single use only. Do not use if pack is damaged.</p> <p>Report any serious incident involving the device to the manufacturer and the competent authority.</p>	<p><b>3. STORAGE CONDITIONS</b> Keep away from direct sunlight and heat. We recommend: - Store at temperature 10-30 °C before use. - Relative humidity: 30-60 % before use. Outer cartons should not suffer physical damage in handling and storage.</p> <p><b>EN 868-5    ISO 11607-1</b></p> <p></p> <p><b>STERIFIT S.r.l.</b> Via dei Martinitt, 3 20146 Milano - Italia info@sterifit.com – www.sterifit.com</p>
---	--	---