



<p>GB INSTRUCTION FOR USE STERIPAPER POUCHES AND REELS</p> <p>1. INSTRUCTIONS FOR USE Use STERIPAPER pouches and reels to pack medical devices to be sterilized by steam or EO gas.</p> <p>Seal the open end(s) with a rotary or impulse sealing machine at the recommended temperature (150-180 °C) and following the instructions of the sealing machine manufacturer.</p>	<p>2. ESTERNAL INDICATORS PRINTED ON THE PRODUCTS:</p> <p>EO gas: Yellow/Brown sterilized Steam: Brown sterilized</p> <p>The packaging is for single use only. Do not use if pack is damaged.</p> <p>Report any serious incident involving the device to the manufacturer and the competent authority.</p>	<p>3. STORAGE CONDITIONS Keep away for direct sunlight and heat. We recommend:</p> <ul style="list-style-type: none"> - Store at temperature 10-30 °C before use. - Relative humidity: 30-60 % before use. <p>Outer cartons should not suffer physical damage in handling and storage.</p> <p>EN 868-5 ISO 11607-1</p> <div style="text-align: right;">  </div> <p>STERIFIT S.r.l. Via dei Martinitt, 3 20146 Milano - Italia info@sterifit.com – www.sterifit.com</p>
---	---	---

<p>GB INSTRUCTION FOR USE STERIPAPER SELF-SEALING POUCHES</p> <p>1. INSTRUCTIONS FOR USE Use STERIPAPER self-sealing pouches to pack medical devices to be sterilized by steam or EO gas.</p> <p>Sealing instructions</p> <ul style="list-style-type: none"> - Unfold flap; - Insert item into pouch; - Remove backing from sealing strip; - Refold along crease pressing down firmly from center to each side to form good seal between adhesive strip and paper and plastic. 	<p>2. ESTERNAL INDICATORS PRINTED ON THE PRODUCTS:</p> <p>EO gas: Yellow/Brown sterilized Steam: Brown sterilized</p> <p>The packaging is for single use only. Do not use if pack is damaged.</p> <p>Report any serious incident involving the device to the manufacturer and the competent authority.</p>	<p>3. STORAGE CONDITIONS Keep away for direct sunlight and heat. We recommend:</p> <ul style="list-style-type: none"> - Store at temperature 10-30 °C before use. - Relative humidity: 30-60 % before use. <p>Outer cartons should not suffer physical damage in handling and storage.</p> <p>EN 868-5 ISO 11607-1</p> <div style="text-align: right;">  </div> <p>STERIFIT S.r.l. Via dei Martinitt, 3 20146 Milano - Italia info@sterifit.com – www.sterifit.com</p>
--	---	---