

GB **INSTRUCTION FOR USE**
VERISTERIL LT POUCHES AND REELS

1. INSTRUCTIONS FOR USE

Use VERISTERIL LT pouches and reels to pack medical devices to be sterilized by hydrogen peroxide, ethylene oxide and gamma ray.

Seal the open end(s) with a rotary or impulse sealing machine at the recommended temperature (120-140 °C) and following the instructions of the sealing machine manufacturer.

2. ESTERNAL INDICATORS PRINTED ON THE PRODUCTS:

VERISTERIL LT:
Hydrogen peroxide: Pink sterilized

The packaging is for single use only.
Do not use if pack is damaged.

Report any serious incident involving the device to the manufacturer and the competent authority.

3. STORAGE CONDITIONS

Keep away from direct sunlight and heat. We recommend:

- Store at temperature 10-30 °C before use.
- Relative humidity: 30-60 % before use.

Outer cartons should not suffer physical damage in handling and storage.

EN 868-5 ISO 11607-1



STERIFIT S.r.l.

Via dei Martinitt, 3
20146 Milano - Italia
info@sterifit.com – www.sterifit.com